LADY-COMP®
BABY-COMP®

Natural contraception

..... the safe cycle computers

Natural family planning

LADY-COMP®
BABY-COMP®

..... a topic which is important to women ..

Contraception

Family planning

safe

natural

simple

fast

without side-effects

..... reliable

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Women's nature: the measure of things

On average a woman can become pregnant on six days per cycle – five days before ovulation and on the day of ovulation.

In order not to become pregnant in this relatively short period of time, many women accept day in, day out considerable impairments of their well-being, they put a heavy strain on their health or their sex life with contraceptive measures.

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With Valley Electronics' cycle computer you do not only acquire a high-quality electronic device, but also the concerted knowledge brought together over 15 years of development work by a highly committed research team.

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you can upgrade your cycle computer with the Baby-Comp programme.

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The daily fertility displays of Lady-Comp and Baby-Comp have a Pearl-Index of 0.7 or to explain it more easily of 99.3 %.

Comparison: The pill has a Pearl-Index from 0.1 to 0.9, the coil from 1 to 3. That is to say that the cycle computers – when safety is concerned – is top.

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And how is the Pearl-Index ascertained?

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it had been more than 700,000 cycles of ten thousands users. In so doing, the requirement on the Pearl-Index was fulfilled.

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In so doing, this resulted in a Pearl-Index of (method safety) 0.7.

* A retrospective study on reliability, acceptance and safety of mini-computers Lady-Comp and Baby-Comp in family planning – 1997.

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Your so-called wake-up or basal body temperature is stored in the computer.

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...you receive your wake-up or basal body temperature required by the computer for evaluation.

However, who gets out of bed directly after the wake-up call?

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The devices calculate the future cycle course from the available values – and resulting from this the fertility.

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The devices control each temperature reading value whether “normal” or whether it was changed by one of the mentioned interruptions. Accordingly it is taken into consideration when any further evaluation of the temperature curve is made – or not. If the temperature reading value would suggest that there is fever, an “F” appears on the display.

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or if you celebrated all night long in order not to enter “abnormal” values.

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El conocimiento básico de la vida y la ...*(¿¿)* fertility capabilities of egg cells and spermatic cells are programmed. If Lady-Comp and Baby-Comp require additional information on your individual cycle, it can be calculated by analysing the morning temperature readings:

- •verde• significa “infertil”
- •rojo• significa “fertil”
- •amarillo• significa aprendizaje y area transitoria.

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Lady-Comp after the coil

Though you can start right away with Lady-Comp if you have a copper coil, it is highly recommended to remove the coil before. If you have a hormone coil, this has to be removed before at any rate. Afterwards you can normally start with your daily temperature readings.

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do without contraceptives on “red” days. However, you can upgrade Lady-Comp subsequently with a great deal of valuable extras, the sort of which Baby-Comp is equipped with. More information on the following pages.

La vida toma su curso,(¿¿)

Life takes its course,
partnership works out fine,
only a child is missing to be
completely happy.

But what can be done
if it does not want to come along -
the domestic bliss?

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sexual intercourse e.g. on the third and on the first day before ovulation or on the day of ovulation – depending on the present individual conception problem.

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After approx. four cycles a print-out makes sense. Our experienced teams in Germany and world-wide would be pleased to consult you individually.

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chances are 50:50. According to a hypothesis the child's sex is also dependent on whether the sexual intercourse which leads to fertilisation occurs on the day of ovulation or before.

Meanwhile practical experience is available on apt predictions of such calculations. According to this, the hypotheses made by Prof. John T. France, University of Auckland, New Zealand (and member of the "Medical Advisory Board of the New Zealand Association of Natural Family Planning" 1988-93) prove to be more correct than reports to the contrary.

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please translate:

“ I wanted to know it – preferably immediately! I really had the feeling that it worked out this time! When Baby-Comp showed a probable pregnancy after the 15th day I was indescribably happy.”

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Esto es llamado infertilidad de mujeres u hombres... (¿?) **if diagnosed by relatively comprehensive tests in a hospital. In all other cases of unwanted childlessness there is no infertility but only a more or less reduced fertility which can result from various reasons.**

With Baby-Comp print-outs of the stored data and our evaluation programme we can ascertain them and we will inform the user.

Observando el ciclo

The first step to face these difficulties to conceive is to observe the cycle:

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“International Meeting on infertility and supportive reproduction technology” in Porto Cervo in Sardinia in 1997.

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please translate:

“Hope and despair –

It was a great relief for my partner and me to receive facts on the cycle and to be able to deal with it accordingly.”

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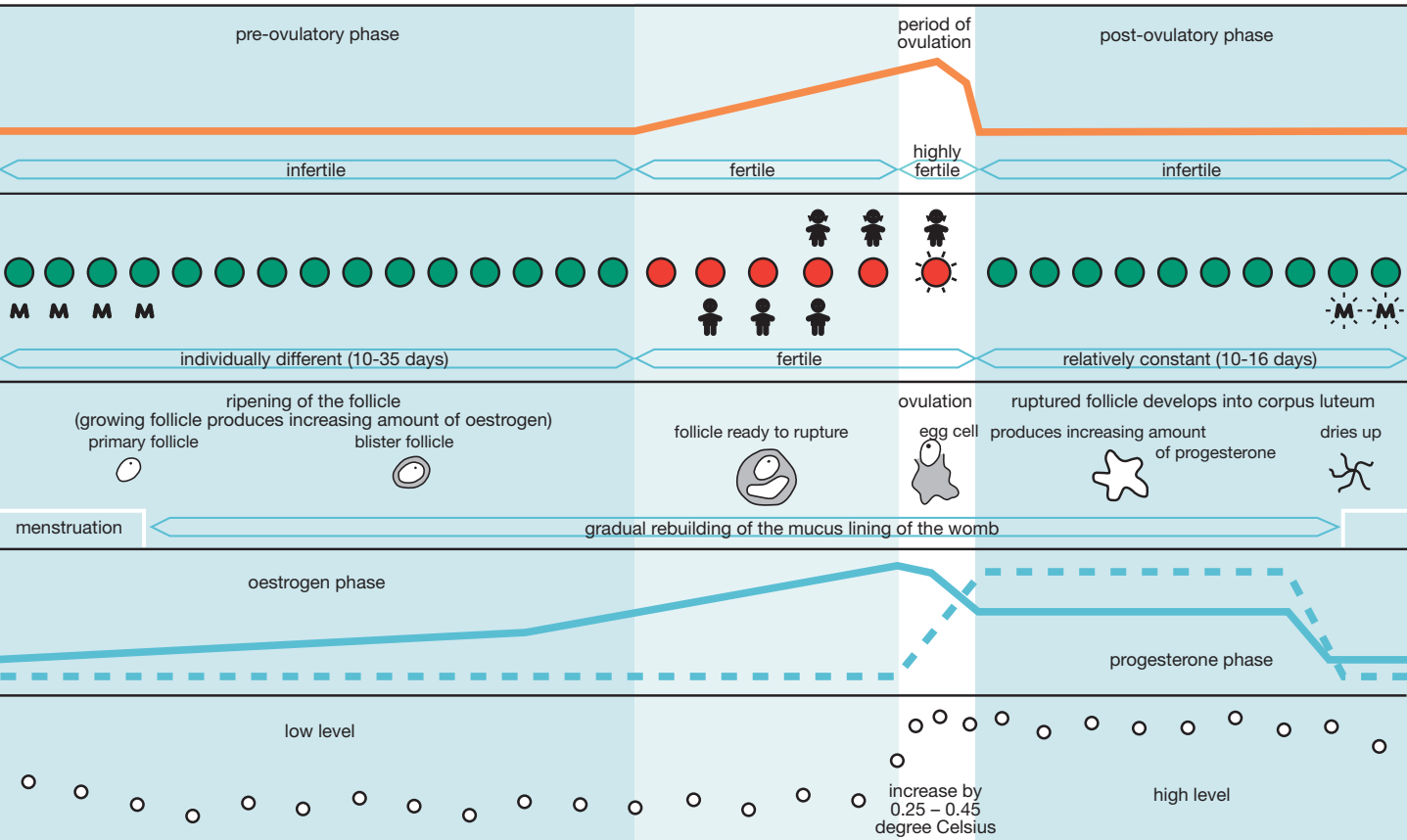
MÉTODO DE TEMPERATURA

in combination with OBSERVANDO MUCUS CERVICAL

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PEARL INDEX

Número de embarazos if contraception in 100 female cycles is made with this method. The lower the value, the higher the safety.

Pearl Index	PRO	CONTRA
1.0 – 5.0	No side-effects, can be used at any time. Protects against infections, such as aids.	On highly fertile days particularly disciplined usage required.
> 1.0	No side-effects, natural way. Women get to know their body. Muy fiable.	Requires discipline. Many application and evaluation regulations. It requires experience to interpret the curves and signs in a correct way.
0	Absolutely reliable.	Operation which can hardly be made undone.

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All data for exact recognition of ovulation is programmed.

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“ In the long run only natural contraception is healthy and without hormonal burden on the environment.”

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“I already have documentation on my cycle. Can I use it?”

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That hardly happens. In order to delete data you have to follow a correct key code which was consciously chosen in such way that data cannot be deleted inadvertently.

“Will my data be lost in case of power failure?”

No. Lady-Comp y Baby-Comp son proveidos de baterías integradas y un cargador. Your data will remain saved indefinitely, date, time and year continue.

Lady-Comp and Baby-Comp are delivered with an integrated battery and a mains power supply for recharging.

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VE Valley Electronics GmbH